




Fraunhofer

IIS

FRAUNHOFER INSTITUTE FOR
INTEGRATED CIRCUITS IIS

DEFIBERIZER

**IMAGE OPTIMIZATION FOR
FIBER-OPTIC IMAGING SYSTEMS**

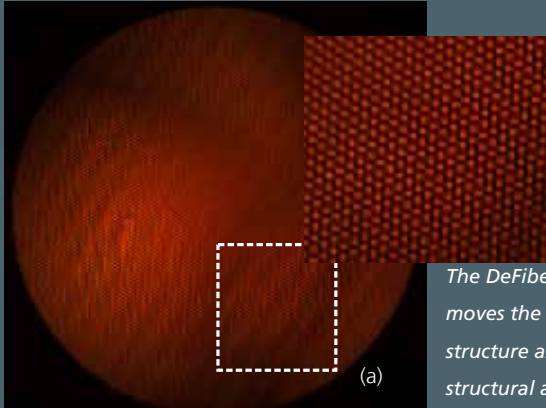

The DeFiberizer software removes the honeycomb structure as well as color and structural artifacts.

For direct visual examination of inaccessible body cavities, semirigid or flexible endoscopes (fiberscopes) are preferably used.


Typical examples in a clinical environment are bronchoscopy, gastroscopy and neurosurgery and, in the technical area, inspection and maintenance of turbines and pipes.

Challenge

Due to their flexible construction and miniaturization, the fiber-optic image conductors employed exhibit a honeycomb interference pattern (a) and poor brightness and color reproduction. These negative effects lower the image quality and, in conventional systems, are usually reduced through defocusing and adaptation of brightness and contrast.

Our Solution

To suppress the honeycomb pattern, all fiber centers in the fiber-optic image (a) are first registered in a calibration step. Direct access to the raw data from the color filter array in the camera enables optimized calculation of individual correction factors for color distribution within the various cells. Together with the information on local adjacency relationships, this forms the basis for subsequent interpolation. With the algorithms described, global shadowing effects and color artifacts can be visibly reduced (b). The software automatically adapts

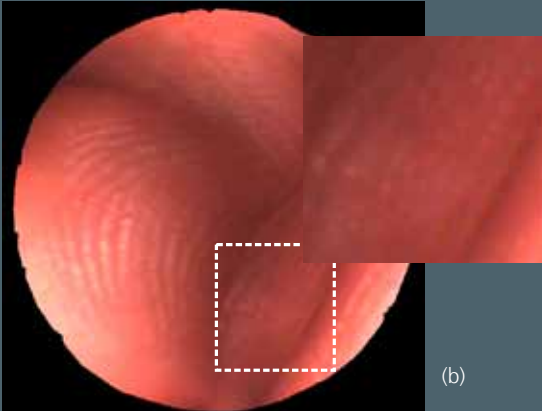

to most combinations of commonly used endoscopes and suitable digital camera modules. Operation from a PC platform additionally allows for acceleration via powerful graphics hardware. Adaption to proprietary or μ C-based platforms possible.

Features

- Use of raw camera data to optimize color display
- Automatic parameterization
- Supports all camera and endoscope types

Specifications

Endoscope

Working diameter: 0.5 – 4 mm

No. of fibers: 3,000 – 30,000

Image conductor: fiber bundle

Camera

Resolution: 640 x 480 – 1024 x 768 (typically 800 x 600)

Digital Interface: FireWire, USB, ...

Software

Modes: grayscale, color, raw data from color filter array

DeFiberizer has not yet been certified as a medical product. Fraunhofer IIS presents this functional prototype with the aim of engaging partners for further development as well as for production and marketing.

WWW.IIS.FRAUNHOFER.DE/MED

***Fraunhofer Institute for
Integrated Circuits IIS***

Executive Director

Prof. Dr.-Ing. Heinz Gerhäuser

Director

Prof. Dr.-Ing. Günter Elst

Am Wolfsmantel 33

91058 Erlangen

Germany

Phone +49 9131 776-0

Fax +49 9131 776-999

info@iis.fraunhofer.de

www.iis.fraunhofer.de

Image Processing and Medical Engineering

Dipl.-Inf. Christian Weigand

Contact

Dr. Christian Münzenmayer

Phone +49 9131 776-7310

Fax +49 9131 776-7309

christian.muenzenmayer@iis.fraunhofer.de